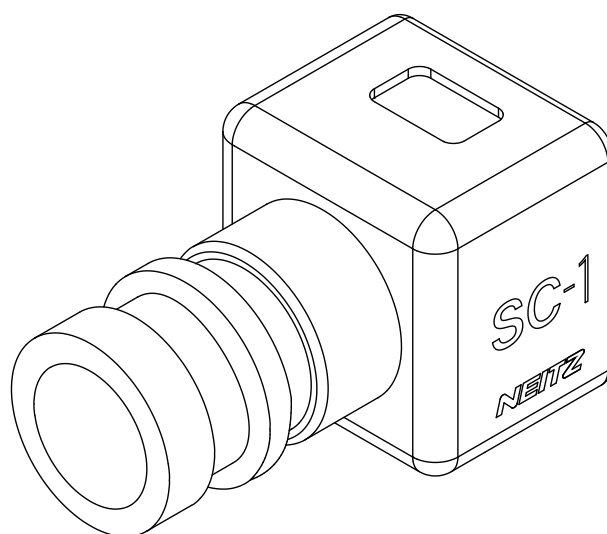
NEITZ

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TD11-13060001-1

Table of Contents

1. Important Information	2
1.1 Intended Use	2
1.2 Symbols	2
1.3 Safety Information	2
2. Compositions and Nomenclature	4
2.1 Composition	4
2.2 Nomenclature and Function	5
3. Operation	6
3.1 Preparation before Use	6
3.2 Operation	7
4. Maintenance	8
4.1 Exterior Cleaning	8
4.2 Lens Cleaning	8
4.3 Replacing Forehead Cushion	9
4.4 Disposal	9
5. Troubleshooting	9
6. Specifications	10
7. Contact Information	11
Annex: Lens Units	11

Thank you for purchasing the NEITZ surgical camera system SC-1. The camera is to attach to the NEITZ spot illuminators and binocular loupes for taking an image of the visual field or illumination field.

Please read this User's Manual carefully before use to avoid unexpected accidents and store it in a safe place for future reference.

1. Important Information

1.1 Intended Use

The Surgical Camera System SC-1 is a device to take an image of the visual field or illumination field. Attached to Neitz Neo Frame or Headband. SC-1 can be used in combination with Neitz spot illuminators and binocular loupes.

1.2 Symbols

The following symbols are used on this manual and/or package to assist you in proper handling and use of the SC-1, and to warn and caution you of potential hazards to yourself and others.

	Consult User's Manual for information on proper use.
	Temperature limits (°C) for storage and transport.
	Relative humidity limits (%) for storage and transport.
	Atmospheric pressure limits (hPa) for storage and transport.
	Fragile. Handle with care.
	Keep dry.
	Serial number.
	Products bearing this symbol must be recycled separately from other household wastes under the WEEE directive
	Manufacturer
	Symbol representing marketing and use in the EU countries are possible.
	To improve distribution efficiency and ensure traceability, the product code and serial number are shown as a GS1 barcode (example shown in figure on left).

1.3 Safety Information

For your safety, please comply with the following precautions.





	Potentially hazardous situation which, if not avoided, could result in death or serious injury.
	Potentially hazardous situation which, if not avoided may result in minor or moderate injury or property damage.
	Prohibited actions (must not be performed)
	Required actions (must be performed)

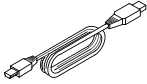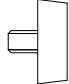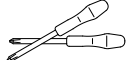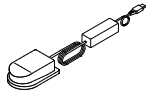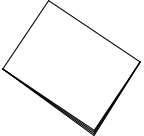
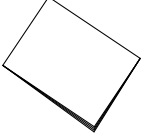
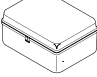
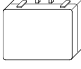
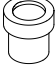
⚠ WARNING	
 Prohibited	In case the cables or cords are damaged, do not use. This may result in electric shock, fire or device failure.
 Prohibited	Do not use in oxygen rich environments, in the presence of propane, gasoline or other combustible gasses, or near dust and debris. Doing so may cause explosion.
 Prohibited	Do not leave the camera turned on for a long period or cover the device during use. Doing so may cause the device to reach high temperatures.
 Prohibited	Do not attempt to disassemble or modify the device. Doing so may result in electric shock, fire or device failure.
⚠ CAUTION	
 Prohibited	Do not use accessories other than the standard accessories supplied by the manufacturer. Doing so may result device failure.
 Prohibited	Do not use in excessively humid or saline environments, or places where the device may be sprayed with water. Doing so may result in electric shock or device failure.
 Prohibited	Do not expose the device to direct sunlight or harmful radiation. Doing so may lead to unexpected heating and result in device failure.
 Prohibited	Do not sterilize the device. Doing so may cause device deformation or failure. Do not immerse in cleaning detergents or rinse with water. Doing so may cause device failure.
 Prohibited	Do not use paint thinners, cleaning agents or boiling water to clean the device. Doing so may cause deformation or device failure.
 Prohibited	Do not put a heavy load on the cables, cords, or plugs. Doing so may result in electric shock or fire.
 Prohibited	Do not pull the cables, cords, or plugs strongly. Do not fold them into a small shape. Doing so may result in failure.
 Prohibited	When handling the USB cable and foot switch cable, avoid the patient and her/his adjacent areas. A contact of the patient with the cables may cause an injury.
 Prohibited	Do not use the foot switch with the converter box hung in midair. The power plug may come off.
 Prohibited	Do not touch the device or cables with wet hands. Doing so may result in electric shock.
 Instruction	Before use, make sure the camera has no damage, no backlash and is firmly attached. If something is wrong with the device, do not use it.
 Instruction	Inserting and removing the shield repeatedly may cause loose attachment due to wear. In that case, contact your local dealer for a replacement. (For Neo Frame type)
 Instruction	Allow the device to return to room temperature before unpacking. Not doing so may cause condensation and lead to device failure.
 Instruction	Unplug from the USB cable if the device is not used for a long period. Not doing so may cause fire or device failure.
 Instruction	Use the attached Carrying case for transport and storage. Not doing so may cause fire or device failure.

2. Compositions and Nomenclature

2.1 Composition

Make sure to match all items in the package with the components shown below and inspect each item for damage. Should any items be missing or damaged, do not use the SC-1 and immediately contact your local dealer.


Component	Neo Frame type	Headband type	Loupe / Illuminator type	Loupe with illuminator type	Drawing
Camera SC-1	1	1	1	1	Se 2.2
Neo Frame	1	—	—	—	Se 2.2
Headband	—	1	—	—	Se 2.2
Hinge 1	—	—	1	—	Se 2.2
Hinge 2	—	—	—	1	Se 2.2
Lens unit	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	
Lock ring	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	1 (attached to SC-1 at the factory)	
USB cable	1	1	1	1	
Fixing screw	—	—	—	1	
Shield	1	—	—	—	
Half moon nose pad	1	—	—	—	
Standard nose pad	1	—	—	—	
Lens cloth	1	1	—	—	
Phillips-head screwdrivers	2	2	2	2	
Strap	1	—	—	—	
Wiring bands	2	2	2	2	
Foot switch	1	1	1	1	

User's Manual (this booklet)	1	1	1	1	
User's Manual (for Capture Software)	1	1	1	1	
Carrying case (Cloth)	1	—	—	—	
Carrying case (Metal)	—	1	—	—	
Package (Blue carton)	—	—	1	1	
Lens unit*	Option				


2.2 Nomenclature and Function

[SC-1 schematic drawings]

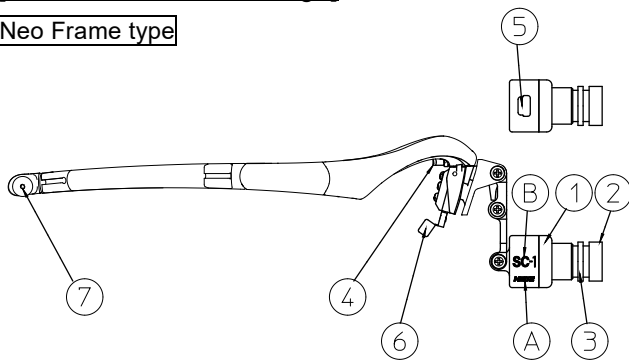
Neo Frame type


Headband type

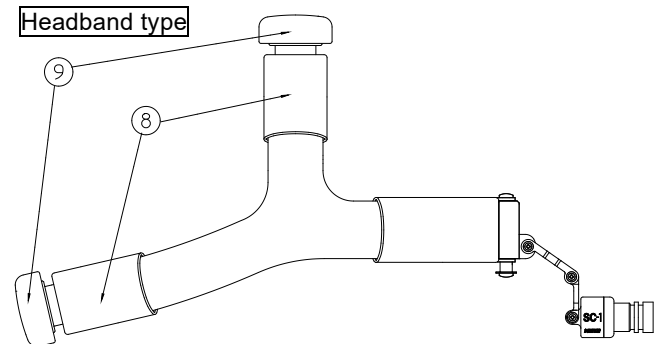Loupe / Illuminator type

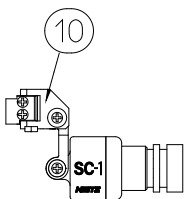Loupe with illuminator type

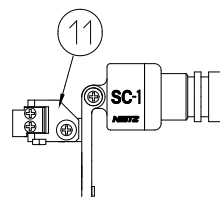Indications on the product and functions

N o .	Indication	Description and function
Ⓐ	<i>NETZ</i>	Manufacturer's Logo
Ⓑ	SC-1	Model Logo
①	Camera SC-1	Incorporates a CMOS camera (hereinafter referred to as "Camera")
②	Lens unit	Turn the Lens unit for focusing.
③	Lock ring	Prevents unintentional movement of the Lens unit after focusing.
④	Forehead cushion	Contacts and guard your forehead.
⑤	USB jack	Accepts the USB cable for connection with your computer.
⑥	Nose pad	The height of the Nose Pad is adjustable to fit to your nose. Two types are available. Select and use according to your preference.

⑦	Strap holder	Eyelets to attach the strap.
⑧	Cloth cover	Covers the Headband mechanism. Do not remove for your safety.
⑨	Size adjustment knobs	Located at the front and rear of the Headband. Knobs to adjust the Headband size.
⑩	Hinge 1	Hinge to attach the Camera to the loupe or illuminator. Adjusts the camera angle.
⑪	Hinge 2	Hinge to attach the Camera to the loupe with illuminator. Adjusts the camera angle.

[Optional accessories]

The following optional accessories to improve usability and functionality are available.


- Lens units (The lens unit included in the delivery kit is the suitable one according to your configurations. If you like some other lens unit, refer to the Annex “Lens Units” and order as an option.)

3. Operation

3.1 Preparation before Use

3.1.1 Attaching to the Loupe or Illuminator

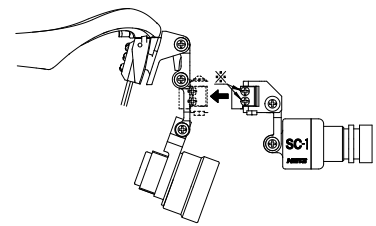Loosen the two screws on the camera hinge (marked with*) and insert the hinge of the loupe or illuminator. Retighten the screws to fix the camera.


3.1.2 Attaching to the Loupe with Illuminator

1) Detach the fixing nut on the back of the illuminator.

2) Attach the illuminator to the hinge of the camera using the fixing screw in the kit.

Place the cord of the illuminator on your left. In case the fixing screw is not fully tightened, rotate the illuminator a bit.

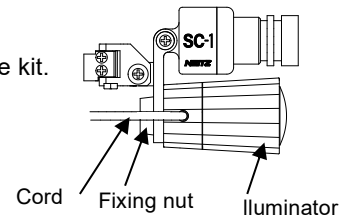
Do not use the illuminator’s fixing nut to fix the camera.

3) Attach to the loupe in the same way as 3.1.1 above.

3.1.3 Wiring for Neo Frame

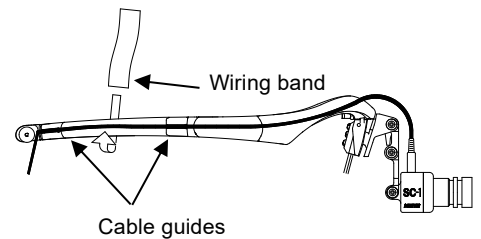
Pass the USB cable through the two cable guides of the Neo Frame and fix the cable at the position in the right illustration using the wiring band. Handle the USB cable with care in order not to apply too much force to it.

3.1.4 Wiring for Headband

Open the stopper button of the of the cloth cover and pass the USB cable through the inside of the cloth cover. Fasten the stopper button.

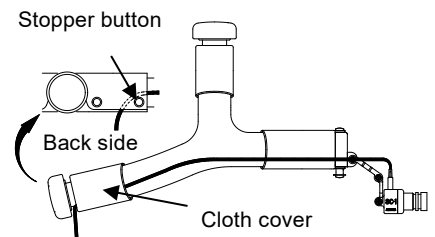Handle the USB cable with care in order not to apply too much force to it.

3.1.5 Connecting to your PC

Make sure to install the Capture Software prior to use. Refer to the User’s Manual of the Capture Software for further information for installation and use.

1) Start up the computer with the Capture Software installed.

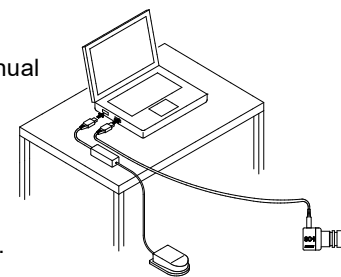
2) Connect the camera to the computer using the attached USB cable.

3) To use the foot switch, connect the foot switch to the PC using its USB plug.

4) Run the Capture Software. Make sure to connect the camera and the PC prior to starting the Capture Software.

Otherwise the camera and software will not work properly. In such case, finish the Capture Software and reactivate after connecting the camera and the PC.

5) Configure the camera settings in accordance with the Capture Software User’s manual.

3.2 Operation

3.2.1 Use of Neo Frame

- 1) Loosen the strap before wearing the Neo Frame.

Adjust the forehead cushion and nose pad to fit best.

The length of the slidable nose pad can be adjusted easily.

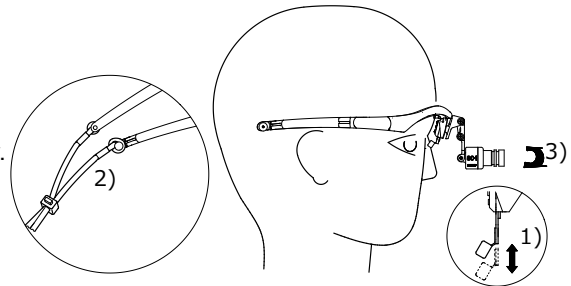
- 2) Tighten the strap's stopper to fix the frame to your head.

- 3) Set the camera to the shooting position.

The position can be set freely by bending the hinges.

* Cord wiring: Do not bend the cord in a sharp angle or pull strongly. Make sure to wire in a circular pattern.

Bundle the cord using the wiring bands as necessary.

3.2.2 Use of Shield

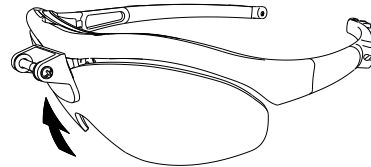
CAUTION

Inserting and removing the shield repeatedly may cause loose attachment due to wear. In that case, contact your local dealer for a replacement. (For Neo Frame type)

- 1) Attach the Shield to the Neo Frame in order to prevent chips, debris or droplets from splashing.

- 2) Insert the Shield into the groove of the frame from below as shown in the illustration in the right.

- 3) Insert the Shield fully until it is seated securely to prevent the shield from falling abruptly.

3.2.3 Use of Headband

- 1) Wear the Headband so that the Camera is centered in front of your face.

- 2) Turn the Size adjustment knob on the top of Headband so that the front cover comes into contact with your forehead and above your eyebrows.

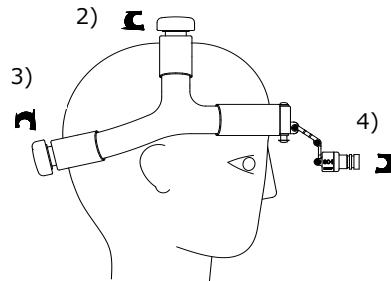
- 3) Turn the Size adjustment knob on the rear of Headband to tighten the Headband for a snug fit.

- 4) Move the lens barrel up and down, left and right to adjust the position of the illumination.

The position can be set freely by bending the hinges.

* Cord wiring: Do not bend the cord in a sharp angle or pull strongly. Make sure to wire in a circular pattern.

Bundle the cord using the wiring bands as necessary.


3.2.4 Adjusting hinge movement

The movement of hinges can be adjusted by tightening or loosening the tension of the screws in the hinge joints.

In case the hinge is hard to move, loosen the joint screws slightly by inserting the two enclosed Phillips-head screwdrivers from both sides and turning counter-clockwise.

Tighten the joint screws securely to restrict the hinge movement and keep the illuminator in place.

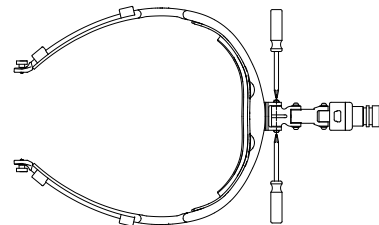
The hinges may loosen with time. Adjust the joint screws regularly.

3.2.5 Use of Stopper

The stopper on the hinge is an auxiliary mechanism to help to return the flipped-up lens barrels to the original position. Follow the instructions below to set the stopper:

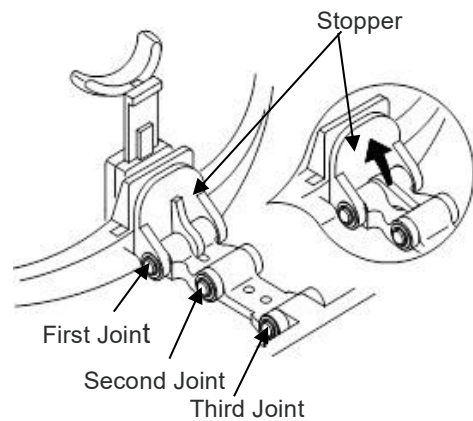- 1) While wearing the Neo Frame with Loupe attached, adjust the lens barrels to the optimum position, and take the Neo Frame with Loupe off of your head.
- 2) Firmly tighten the screws on both sides of the second and third hinge joints so that these joints can hold secure as is. Tighten the screws on the first joint only slightly, and loose enough but secure enough to hold the lens barrels in a stable position.
- 3) Flip up the lens barrels and pull the stopper upward by 90 degrees or more.

- 4) Put on the Neo Frame with Loupe attached again while keeping the condition indicated in (3) above, and slowly move the lens barrels downward. When the lens barrels reach the optimum position as adjusted in (1) above, flip up the lens barrels once again and take the Frame with Loupe off of the head.
- 5) Tighten the first joint screws slightly but not so firmly so that it can work as the pivot for flipping-up the barrels. The first joint screws should be tightened looser compared to these of the second or third ones, but not too loose. The setting of stopper is completed by following the above procedures. The lens barrels will now retain the optimum position whenever they are flipped downward from the up position.

If the lens barrels flip downward too much and the stopper is pushed into the hinge, reset the stopper once again with the same procedure as mentioned above. If the stopper is not used, push it completely into the hinge.

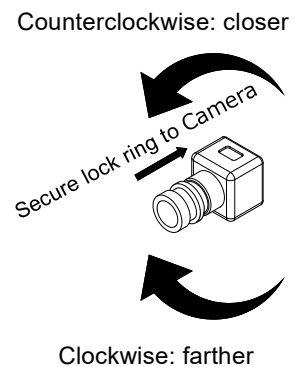
3.2.6 Focusing

Bring the camera into focus by turning the lens unit while viewing the PC display.

It is recommended to seek the help of an assistant, however, if it is difficult, adjust the focus simulating the working distance or position beforehand.

Focus on distant objects by turning the lens unit clockwise, and focus on near objects by turning it counterclockwise.

After focusing, secure the lock ring to the Camera to fix the lens unit.

3.2.7 Shooting

Shoot according to the User's manual of your Capture Software.

4. Maintenance

4.1 Exterior Cleaning

Wipe the surface using a dry lint-free cloth or cloth soaked with diluted neutral detergent and wipe dry.

For disinfection, wipe the exterior using a soft cloth soaked with ethanol.

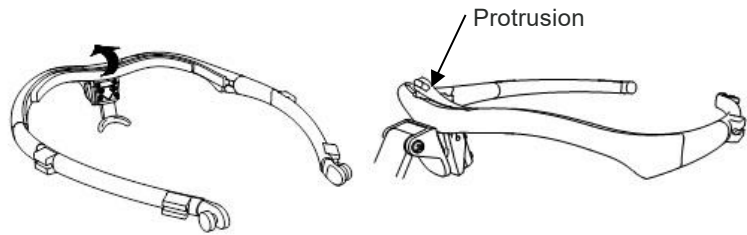
4.2 Lens Cleaning

If dust or other foreign matter adheres to the lens, remove with an air blower or the like. If the foreign matter cannot be removed with an air blower, wipe gently using a soft cloth soaked with diluted neutral detergent and wipe dry.

4.3 Replacing Forehead Cushion

1) If the Forehead cushion wears out or deteriorates, replace it with a new one which is available as an optional accessory.

2) To remove the used Forehead cushion on the frame, pull the protrusion on the center of the cushion out of the Fixing hole on the frame while expanding the frame slightly, and then pull the end of Forehead cushion out of the pivot space completely from the frame.

Repeat on the other side.

3) When mounting a new Forehead cushion, insert the both ends of the new Forehead cushion into the right and left pivot spaces and insert the protrusion on the Forehead cushion into the hole on the frame. Since there is an up and down edge on the Forehead cushion mount it in the proper orientation.

4) In case the Forehead cushion comes off abruptly, attach in the same manner.

4.4 Disposal

Dispose of the camera and its accessories in accordance with your local regulations and/or environmental guidelines.

5. Troubleshooting

If you experience any problems using the SC-1, refer to the chart below for possible solutions.

If the problem cannot be solved, stop using and immediately contact your local dealer or Neitz directly.

Problem	Possible Cause	Solution
Capture Software does not start.	Camera is not connected to your PC.	Connect with the USB cable.
No image displayed.	Some PCs may supply no or insufficient electric power.	Use a PC with power supplying capability.
Blurred or uneven image.	Dust or other foreign matter adhered to the lens unit.	Clean the lens unit.

6. Specifications

Product specifications

Product name	Surgical Camera System SC-1
Compatible with	Neitz Binocular Loupes BLS-1, BLS-2, BLS-3, BLD-3 Neitz Binocular Prismatic Loupes BLP-4, BLP-6 Neitz Spot Illuminators NSI-III, NSI-X
Image sensor	Approximately 2.13 Megapixels, 1945 (H) x 1097 (V) CMOS sensor
Transfer method	Progressive
Shutter system	Rolling shutter
Output image format / Max. frame rate	MJPEG / 30 fps
Sensitivity	1300 mV 30 fps (Standard value: F5.6)
Driver	Not needed. USB Video Class compatible
Interface	USB 2.0 Mini-B
Main IPS functions	Adjustment: exposure (auto & manual), color temperature (auto & manual), gain, saturation, sharpness, and gamma correction.
Power supply	Power supply voltage 5.0 V (USB bus power) Maximum power consumption approximately 200mA
Camera properties	Brightness, contrast, hue, vividness, definition, gamma, gain, white balance, backlight correction
Camera dimensions and weight	20 mm x 20 mm x 23.4 mm (excluding protrusions), 12 g
Safety standard	IEC 62368-1:2018
Environmental standard	RoHS Directive (2011/65/EC)

Use environments

Temperature/Relative humidity/Atmospheric pressure	+10 - +35°C/30 - 90% (no condensation)/800 - 1060hPa
--	--

Storage and transport environments

Temperature/Relative humidity/Atmospheric pressure	-10 - +55°C/10 - 95% (no condensation) /500 - 1060hPa
--	---

7. Contact Information

If you have any questions or need technical support, contact your local dealer or Neitz located at the following address.

In the event of any serious incident that has occurred in relation to this product, please report the incident to the manufacturer, its authorized representative in Europe shown below and to the competent authority of your country of residence.

[LOCAL DEALER]


Neitz Instruments Co., Ltd.

4F Ichibancho Court, 15-21, Ichibancho, Chiyoda-ku

Tokyo 102-0082, Japan

Phone: +81-3-3237-0552, Fax: +81-3-3237-0554

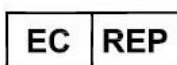e-mail: neitz-global@neitz.co.jp, URL: <https://www.neitz.co.jp/en/>

Medical Device Safety Service GmbH

Schiffgraben 41, 30175 Hannover, Germany

Annex: Lens Units

The lens unit included in the delivery kit is the suitable one according to your configurations. Refer to the table below for details. If you like some other lens unit, order as an option.

Device	BLS-1	BLS-2	BLS-3	BLD-3	BLP-4	BLP-6	Others
Attached lens	f8 lens	f16 lens	f16 lens	f12 lens	f16 lens	f25 lens	f16 lens

Lens unit	f8 lens	f12 lens	f16 lens	f25 lens
Field of depth*	125 mm	60 mm	80 mm	25 mm
Angle of view	145 mm × 260 mm	90 mm × 160 mm	70 mm × 125 mm	45 mm × 75 mm
Shooting distance	200 mm to 600 mm from front of SC-1	200 mm to 600 mm from front of SC-1	200 mm to 600 mm from front of SC-1	200 mm to 600 mm from front of SC-1

* Specification at 400 mm from front of SC-1.

As part of our policy of continued product improvement, we reserve the right to alter and/or amend specifications at any time without prior notice.

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Neitz Instruments Co.,Ltd.